

# Face Shield.

Assembly instructions.

After use, the individual parts of the shield can be cleaned with glass cleaner or disinfectant.

## CAUTION!

Thinner is not suited for the cleaning of the face shield.

You can find more information about the product here:  
[www.marbach.com/faceshield-assembly](http://www.marbach.com/faceshield-assembly)

## Face Shield.

Assembly instructions.

### SCOPE OF DELIVERY:

**Attention!** Please disinfect all parts before assembly.

1 x Headband  
1 x Schield  
1x Fastener band made of rubber

### OPTIONAL USE:

1 x Rubber band (Ø 80 mm)

### APPLICATION AREA.

In all areas. Face shield for shielding against droplets in the air.

### TIP:

For sweat absorption, a cloth can be attached to the headband with a paper clip.



### COMPANY NAME AND FULL ADDRESS OF THE MANUFACTURER

Karl Marbach GmbH & Co. KG  
Karl-Marbach-Straße 1  
74080 Heilbronn  
Germany  
[info@marbach.com](mailto:info@marbach.com)

## Face Shield.

Assembly instructions.

### ASSEMBLY INSTRUCTIONS.

- Step:**  
Plug the headband and shield together.
- Step:**
  - Insert the Fastener band made of rubber from the inside through the outside flap.
  - Hang the Fastener band in the U-shaped flap.
- Step:**  
Put the Fastener band on the other side through the rectangular tab. Fastener band can be adjusted individually.
- Optional:**  
Optionally, a further rubber band (Ø 80 mm) can be fixed to the bottom of the shield. Thus the mask covers the face even better.

